manual

TENS machine

Transcutaneous Electrical Nerve Stimulator

KTR-2610

Item: 47956

EN

NO

SV

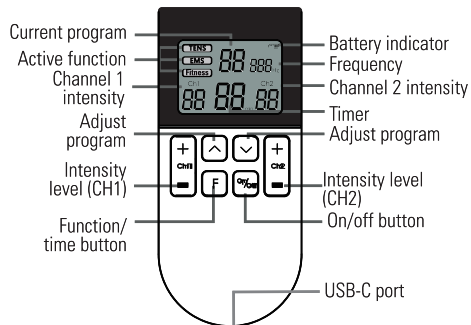
www.rubicson.com
Box 50435 Malmö
Sweden
2024-02-29

2862

Rubicson®

Overview

Specifications

Functions:	TENS, EMS and Fitness
Intensity levels:	16 (1-100 Hz)
Timer:	15/30/45 min
Charging:	USB-C (DC) 5 V/180 mAh
Battery type:	Lithium (3.7 V, 250 mAh)
Safety class:	Class II Type BF

Impulse frequency:	TENS: 20-100 Hz
	EMS: 1-15 Hz
	Fitness: 2-16 Hz
Pulse width:	TENS: 120 µs
	EMS: 200 µs
	Fitness: 200 µs
Impulse waveform:	Square wave
Size:	122x55x16 mm (main unit)
	50x50 (electrode pads)

Environment for use

Temperature:	+5 °C to +40 °C
Humidity:	15-93% RH
Barometric pressure:	700-1060 hPa

Environment for storage

Temperature:	-25 °C to 55 °C
Humidity:	15-93% RH
Barometric pressure:	700-1060 hPa

Included

TENS/EMS main unit, 4x electrode pads, electrode wire, USB-A to USB-C charging cable (50 cm), manual

Charging

Connect the included USB-C cable to the USB-C port and a USB power adapter. The battery indicator shows the status of the battery and when it is fully charged.

Use

Connect the electrode wires to the electrode pads and the main unit. Attach the pads to the part of your body you wish to massage. The skin must be dry and clean.

Turn the machine on by pressing the On/off button. The TENS starts the first program automatically. Press the Function/time button to switch between TENS, EMS or Fitness.

TENS (prg. 1–20) is intended for use for general pain relief associated with sore and/or aching muscles.

EMS (prg. 1–20) is intended for relaxation of muscle spasms, increasing local blood circulation and prevention of disuse atrophy.

Fitness (prg. 1–10) is intended as a complement for abdominal toning and strengthening.

Press the Adjust program buttons to change between the programmes. Press the Intensity buttons (CH1, CH2) to adjust the intensity of each pair of electrode pads. Press and hold the Function/time button to activate the timer. Press again to set the timer to 15, 30 or 45 minutes.

Notes

Only attach the electrode pads to dry skin. The electrode pads can be used up to 80 times and can be stored for 24 months.

Maintenance

Always turn the machine off and disconnect the electrode wires before cleaning the main unit or the electrode pads.

Clean the electrode pads after each use by wiping them with a damp and soft cloth or towel. Add a small amount of medicinal alcohol (75% concentration) if the pads are particularly dirty. The main unit can be cleaned with a soft cloth or towel.

Charge the machine if it has not been used for over a month.

If the main unit has been stored at the minimum/maximum storage temperature/humidity, wait 30 minutes before use.

Do not use the machine in temperatures above/below the recommended working environment as it may drastically decrease the performance of the main unit and its battery.

Store the TENS machine in a well-ventilated dry place, away from water, strong electromagnetic radiation, high temperatures, direct and strong sunlight.

Safety information

Attention

 Do not use high-frequency surgical equipment and the TENS machine at the same time. This might cause burns on the part where the electrode pad is pasted to, or cause damage to the machine.

If the TENS machine is used in the vicinity (1 m) of a short wave or microwave therapy stimulator, the output of the machine might become unstable.

Do not place the electrode pads close to the chest or thorax. Risk for cardiac fibrillation.

Do not modify the TENS machine without authorization of the manufacturer.

The battery should be replaced by professional maintenance staff.

Do not use the TENS machine and high-frequency surgical ME equipment at the same it. This might cause burns on the part where the electrode pad is pasted to, or cause damage to the machine.

Do not place the electrode pads across or through the head, directly on the eyes, covering the mouth, on the front of the neck, (especially the carotid sinus), or from electrode pads placed on the chest and the upper back or crossing over the heart.

Do not inhale or swallow small parts.

Prevent sharp parts from damaging the product.

Do not use accessories not specified by the manufacturer.

⚠️ Contraindications

- ❗ Consult a doctor before use if you suffer from very sensitive skin, heart disease, abnormal blood pressure, malignant tumours, cerebrovascular disease, epilepsy, acute disease currently under doctor's treatment, or if you are having your menstrual period or are pregnant.
- ⊘ People who are not sensitive to or do not feel pain or heat on the skin should not use the TENS machine. It can lead to injuries.

It is prohibited to use when bathing, sweating and sleeping.

The patient with cerebral hemorrhage: It should be disabled in unsteady phase; the person who has sequela must be used under supervision of doctors.

It is contraindicated for use by persons who has purulent inflammation, acute blood poisoning and continuous hyperpyrexia.

It is contraindicated for use by persons who have acute cardiovascular and cerebrovascular diseases.

Stop using the machine and consult a doctor if you at any point feel unwell during use.

⚠️ Precautions

- ⊘ Do not place the electrodes near the heart, head, eyes, front neck (especially the carotid artery), lower back, oral cavity or pudendum. Do not use if you have a skin disease.

- ⊘ Do not move the electrode pads during use. Turn off the machine before detaching the pads and moving them to a new spot as it might cause unpleasant stimulation.

Do not use the machine on children or people with no ability to express their own consciousness.

Stop using the machine and consult a doctor if you at any point feel unwell during use.

Unplug the TENS machine from the power supply when you are done using the product.

Do not use the TENS machine along with other medical electronic stimulators such as a cardiac pacemaker, artificial heart-lung which is used to maintain life, and electrocardiograph. This can lead to danger.

Do not use the TENS machine in direct sunlight, high heat, inflammable environments, electromagnetic radiation and humidity.

Do not disassemble, repair or modify the TENS machine, it can cause failure or you can get an electric shock.

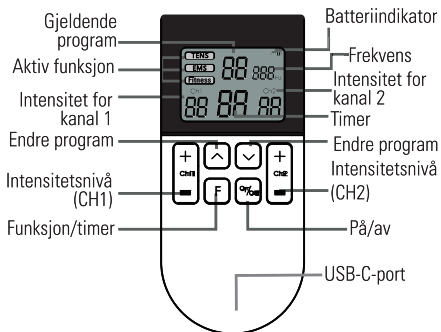
Place the electrode pads where it is easy to move in an emergency.

Check the equipment before each use to avoid the exposed wires caused by accidental damages or other reasons.

Dust may affect the performance of the TENS machine, use a dry soft cloth to clean the machine when needed.

Make sure the electrodes are not loose before use.

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Symbols

Graphic symbol	Paraphrase
	Product batch number tag
	Product serial number tag
	Manufacturer
	Production date
	Precautions
	EU representative
	Warning against danger
	Applies part of type BF
	Class II equipment
	Indoor use only
	Prohibition (things that are not allowed to do)
	Constraint (things that must be observed)
	Keep out of the sun
	Keep away from rain

	Contact local authorities to determine the proper method of disposal of potentially bio hazardous parts and accessories.
	CE symbol and Notified body code
IP22	Dustproof and waterproof grade. It can prevent solid object larger than 12 mm from intruding, and when tilting for 15 degrees, it can still prevent water from intruding, and prevent harmful effects.
	This symbol is used to direct the user to refer to documentation for additional information regarding the system use or description.
	Medical device

Executive standards

The product conforms to the following standards and laws:

1. IEC 60601-1:2005+A1: 2012 Medical electrical equipment-Part 1:
General requirements for basic safety and essential performance
2. IEC 60601-1-11: 2015 Medical Electrical Equipment - Part 1-11:

General Requirements For Basic Safety And Essential Performance - Collateral Standard: Requirements For Medical Electrical Equipment And Medical Electrical Systems Used In The Home Healthcare Environment

3. IEC 60601-2-10: 2013 Medical electrical equipment - Part 2-10: Particular requirements for the safety of nerve and muscle stimulators

4. IEC 60601-1-2: 2014 Medical Electrical Equipment - Part 1-2: General Requirements For Basic Safety And Essential Performance - Collateral Standard: Electromagnetic Disturbances - Requirements And Tests

Appendix EMC declaration

The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the EQUIPMENT should assure that it is used in such an environment.

The Transcutaneous Electrical Nerve Stimulators is suitable for use in a professional health care environment, not including areas where there are sensitive equipment or sources of intense electromagnetic disturbances, such as the RF shielded room of an imaging system magnetic resonance imaging, in operating rooms near active AF surgical equipment, electrophysiology laboratories, armored rooms or areas where short wave therapy equipment is used.

Do not use the system around strong electric filed, electromagnetic filed (e.g. MRI scan room) and mobile

wireless communication devices. Using the device in an improper environment may cause malfunction or damage.

The compliance with EMC and EMI regulation cannot be guaranteed by the use of modified cables or those which does not comply with the same standards under what the equipment was validated.

The system must not be used adjacent or supported by other equipment.

The recommendations of this manual must be followed.

Do not use accessories, transducers, internal parts of components and other cables other than those previously specified by the manufacturer. This may result in increased emission or decreased electromagnetic immunity and result in improper operation.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30cm to any part of the ultrasound system, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

To maintain basic safety in relation to electromagnetic disturbances during the expected service life, always use the system in the specified electromagnetic environment and follow the maintenance recommendation described in this manual.

The following tables provide information on compliance of the equipment according to the standard EN 60601-1- 2:2015.

Table 1 - Compliance class

Emissions test	Compliance	Electromagnetic Environment and Guidance
RF emissions CISPR 11	Group 1	The equipment uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	

Harmonic emissions IEC 61000-3-2	Class A	The equipment is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Table 2- Compliance standards

Phenomenon	Basic Standard of	Immunity Test	Level of
Electrostatic discharge	IEC 61000-4-2	±8 KV contact ±2 KV, ±4 KV, ±8 KV, ±15KV air	±8 KV contact ± 2 K V, ± 4 K V, ± 8 KV, ±15KV air
Radiated RF EM fields	IEC 61000-4-3	3V/m 80 MHz-2.7 GHz 80% AM at 1 KHz	3V/m 80 MHz-2.7 GHz 80% AM at 1 KHz
Proximity fields from RF wireless communication equipment	IEC 61000-4-3	See table	See table

Electrical Fast/ Transients bursts	IEC 61000-4-4	±1 KV 100 KHz repetition frequency	±1 KV 100 KHz repetition frequency
Conducted disturbances induced by RF fields.	IEC 61000-4-6	3 V 0.15 MHz-80 MHz 6 Vm in ISM bands between 0.15 MHz and 80 MHz 80% AM at 1 KHz	3 V 0.15 MHz-80 MHz 6 Vm in ISM bands between 0.15 MHz and 80 MHz 80% AM at 1 KHz
Rated power frequency magnetic fields	IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz

**Table 3 - Test specifications for ENCLOSURE
PORT IMMUNITY to RF wireless
communications equipments**

Test Frequency (Mhz)	Band (MHz)	Service	Modulation	Maximum Power (W)	Distance (m)	Immunity Test Level (V/m)
385	380- 390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27
450	430- 470	GMRS 460, FRS 460	FM ± 5 KHz deviation 1KHz sine	2	0.3	28
710		LTE 13, 17 Band	Pulse modulation 217 Hz	0.2	0.3	9
745	704-					
780	787					

810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE 5 Band	Pulse modulation 18 Hz	2	0.3	28
870						
930						
1720	1700-1900	GSM 1800, CDMA 1900, GSM 1900, DECT, LTE 1, 3, 4, 25 Band, UMTS	Pulse modulation 217 Hz	2	0.3	28
1845						
1970						
2450	2400-2570	Blue-tooth, WLAN 802.1.1 b/g/n, RFID 2450, LTE 7 Band	Pulse modulation 217 Hz	2	0.3	28
5240	5100-5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
5500						
5785						

**Shenzhen Kentro Medical Electronics
Co.,Ltd**

2nd Floor, No. 11, Shanzhuang Road, Xikeng
Village, Yuanshan Street, Longgang District,
Shenzhen City, Guangdong Province, China [Tel]:
+86(755)33825998, [Fax]: +86(755)33825996

WellKang Ltd (www.ce-marking.eu)

Enterprise Hub, NW Business Complex,
1 Beraghmore Road, Derry, BT48 8SE,
Northern Ireland